HZ-3872

Girta Next

Electric Beauty Bed / Camilla eléctrica de estética / Table d'esthétique électrique /
Lettino Elettrico Beauty / Elektrische Kosmetikliege

EN	2
ES	6
FR	10
IT	14
DE	18

DDUUEETT EUROPE would like to thank you for choosing Sonia products. We hope that this instruction manual will help you get the most out of this product.

If you have any doubts regarding its use and/or operation, please contact your distributor.

BEFORE USE

- Please read the entire instruction manual carefully before using the product. Take note of any warnings included in this manual.
- Always keep this instruction manual handy and if you give the device to another person, hand over the manual as it is an essential part of the product.
- Some components of the product may differ from the image that appears in catalogues, websites, brochures, etc.
- In order to enhance the product, the manufacturer reserves the right to modify its features and/or components without prior notice.

CORRECT USE OF THE DEVICE

- This product has been designed to be used by beauty professionals at beauty salons and should not be used for other purposes.
- Please note that any misuse of the device entails the loss of the guarantee.
- Do not make any modification to the device without the authorisation of DDUUEETT EUROPE SL.
- DDUUEETT EUROPE SL shall not be held liable for any material or personal damage due to the misuse of the product.

PRECAUTIONS

- Before using it for the first time and after each use, check the device and the mains cable for any damage.
- Unplug the device from the mains before starting to clean it or undertaking any type of maintenance.
- Do not use the product outdoors or in places where it may be affected by humidity or inclement weather.
- If the power supply cable is defective, unplug the appliance immediately and inform your dealer or specialised technician.
- Position the power supply cable properly so that people cannot trip over it and make sure you plug it into an earthed socket.
- At the end of the working day, disconnect the device and unplug the power supply cable from the mains.
- If you detect any anomaly in the operation of the product, please contact your distributor or specialised technician.

SAFETY INDICATIONS

- The device and accessories should be kept out of the reach of children.
- Only plug the device into a properly installed socket. The local mains voltage must correspond to the technical specifications of the device.
- The power socket must be accessible in case you have to quickly unplug the device at any given time.
- Always pull out the actual plug and not the mains cable.
- Never tamper with any electrical component or open up/repair any component of the device.
- Only your distributor or specialised technician should repair the device.
- Do not switch on the device if there is visible damage to it or to the mains cable.
- Never touch the device or the mains cable with wet hands.
- Never expose your hands or any part of your body to any hole or vacuum on the bed while it is in operation.


CONTENTS

- Electric Beauty Bed x 1 unit
- Headrest x 1 unit
- Armrests x 2 units
- Instruction manual x 1 unit


TECHNICAL SPECIFICATIONS

Dimensions	179x84,5x61-91 cm
Net weight	70 kg
Max. Load	200 kg
Motors	4
Upholstery	PU
Voltage	220-240 V
Input frequency	50/60 Hz
Includes	- Hand and Foot controller - Heating - Airbag backrest - LED light on the base - 5 memorised and customisable positions - 2 rear wheels for easy manoeuvrability

SKETCH AND DIMENSIONS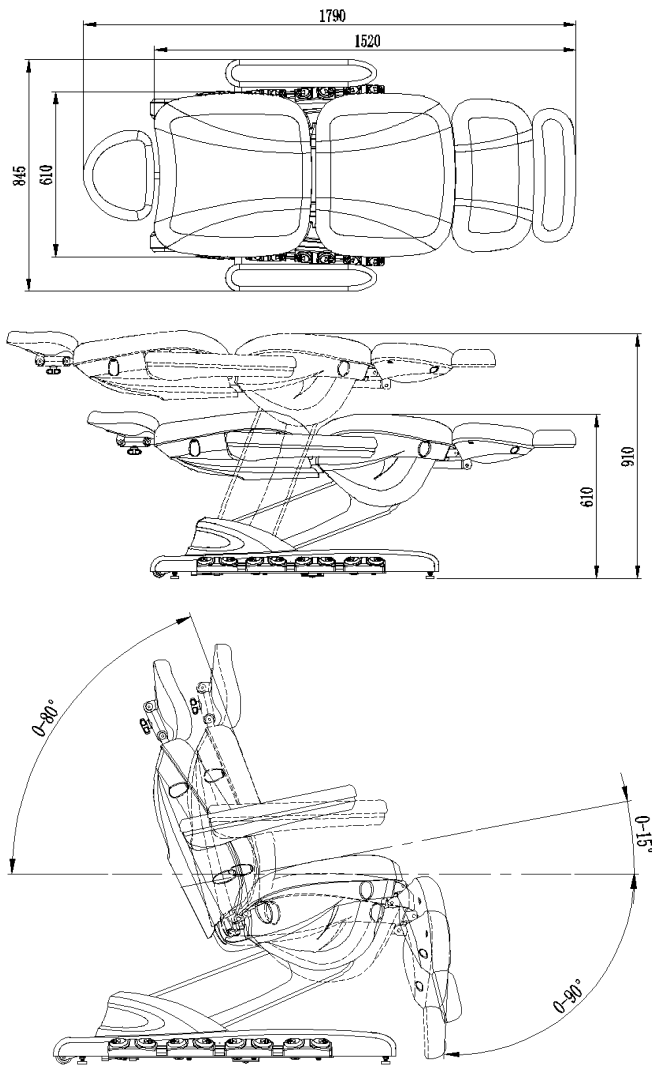


INSTRUCTIONS FOR ASSEMBLY

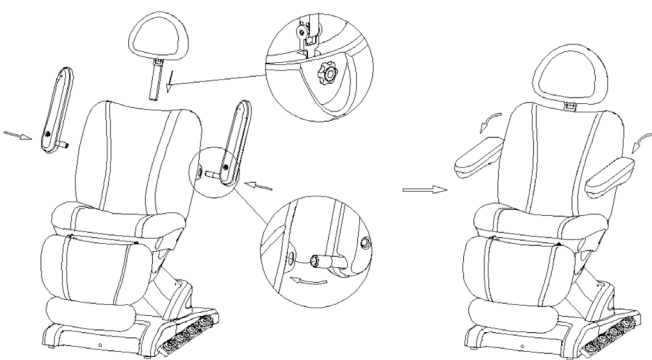
- Unscrew the clamping knob and insert the headrest into the backrest of the main body of the bed. Adjust the headrest to the desired height and turn the knob clockwise.
- Put the armrest in a more or less vertical position, and insert it into the internal pin on the backrest of the main body of the bed. Find the exact position so that the arm connects perfectly and can be coupled correctly.
- Turn the armrest 90° anticlockwise by placing it horizontally.
- Perform the same procedure on the 2 armrests.

OPERATION

- Connect the bed to an electrical outlet using the power cord.
- Turn the ON/OFF switch on the back of the bed to ON. The bed will then be ready for operation via the integrated foot control.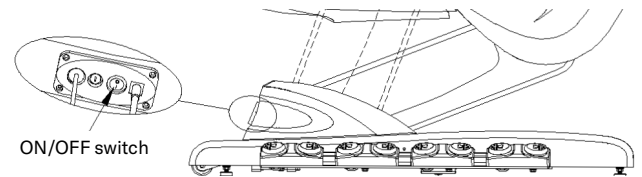


- Hand controller

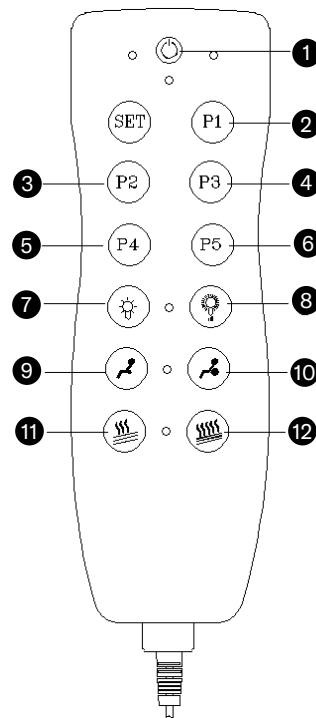- Press the button 1 to activate the control knob. The red indicator will light up and all its functions can be used.
- Press the button 1 again and the red indicator will turn off at the same time as the bed adopts the "chair" position for the user's comfort.
- While the bed is moving towards the "chair" position, press the button 1 again or any button on the integrated foot control and the bed will stop at the position at that time.
- Press the buttons 2 - 6 to place the bed in one of the 5 factory-stored positions.
- Press the button 7 repeatedly to change the colour of the LED lights on the base. After five presses, it will enter cyclic colour mode, and with one more press it will turn off.
- Press the button 8 to adjust the power of the LED lights on the base. Pressing repeatedly will decrease the brightness until reaching a minimum level, after which it will begin to increase again.
- Press the button 10 to inflate the back cushions.
- Press the button 9 to remove air from the backrest cushions.


- Press the button **11** and the heating will start at normal level. The indicator will light up green. Press the button **11** again and the heating will stop. The green indicator will turn off.
- Press the button **12** and the heating will start at high level. The indicator will light up red. Press the button **12** again and the heating will stop. The red indicator will turn off.
- Set any of the 5 factory-stored positions to a new position:
 - Once the bed is in the desired position, press the SET button until the blue indicator lights up.
 - While the blue indicator is lit, press one of the five buttons (P1, P2, P3, P4, P5) for where you want to memorise the new position.
 - The new position will be memorised in the selected button.

• Integrated foot control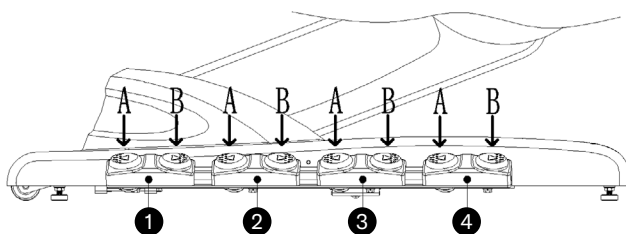


- Adjusting height

- Use buttons A and B of the foot control **1** to adjust the height of the bed.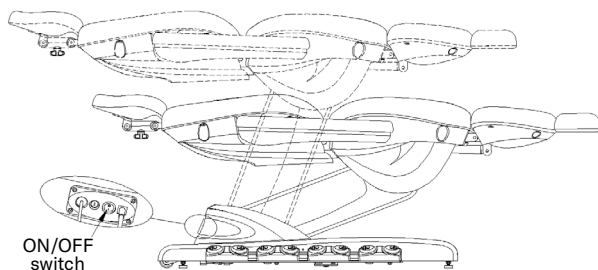


- Adjusting backrest tilt

- Use buttons A and B of the foot control **2** to adjust the tilt of the backrest.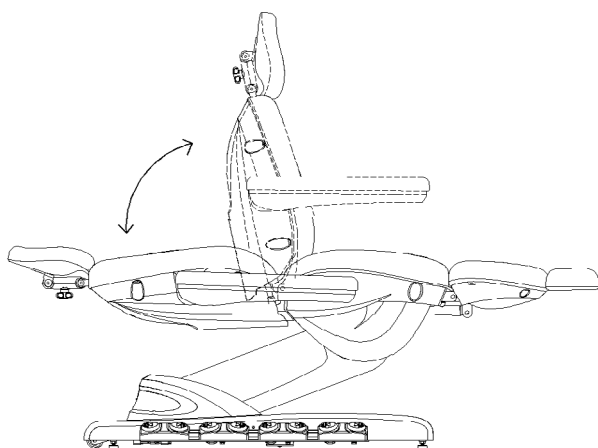


- Adjusting seat tilt

- Use buttons A and B of the foot control **3** to adjust the tilt of the seat.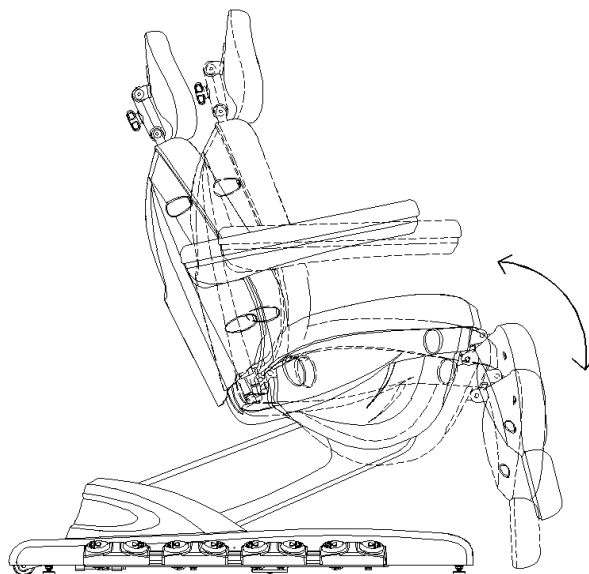

- Adjusting footrest tilt


- Use buttons A and B of the foot control **4** to adjust the tilt of the footrest.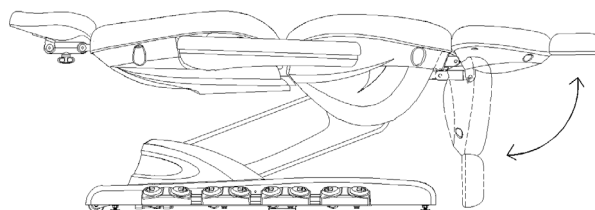

• Manual functions

- Adjusting headrest


- Unscrew Knob A to adjust the height.
- Move the headrest to achieve the desired height and then screw Knob A tight.
- Unscrew Knob B to adjust the headrest in terms of tilt and/or moving its position forwards or backwards.
- Move the headrest to achieve the desired position and then screw Knob B tight.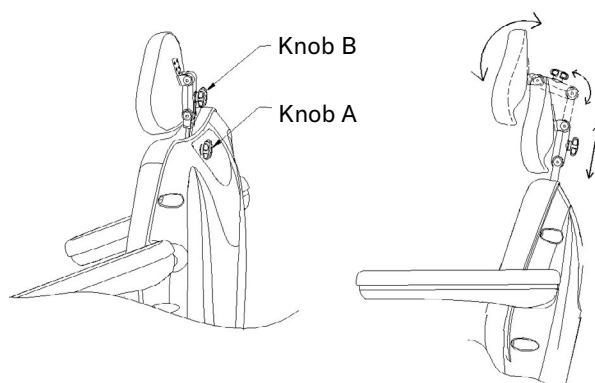

- Adjusting footrest

- Unscrew the clamping knob to adjust the footrest lengthwise.

- Then screw the clamping knob tight.

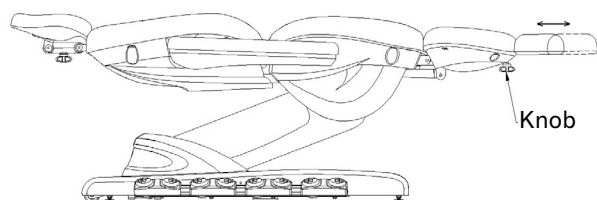CLEANING AND MAINTENANCE

- Before cleaning the device, unplug it from the mains.
- Clean the device with a soft, dry cloth. Do not use liquids to clean it.
- For maintenance or repairs of any part of the product, please contact your distributor or specialised technician.

RECYCLING

If you need to dispose of the product or any of its parts or accessories, please check and respect the laws and regulations applicable to waste management and environmental protection.

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