

1025 series 1026 series Stool

Welcome to use our products. A0
Please read the user manual carefully before using the products.
Our company reserves the right to improve design and modify
the products without prior notice.

1. About This Manual

This manual shall only be used for the reference of operation of this particular model. If the manual is used for other purposes, we will not undertake any according consequences and responsibility. All information of this manual shall be protected by copyright and shall not be copied and translated in any part of this specification without the written consent of manufacturer.

2. Warning

- Do not stand on the product.
- Do not allow children play with the product.
- Do not place the equipment in damp places such as bathroom, do not expose to direct sunlight.
- Do not use this product in the places where you are painting/spraying paints.

3. Precautions


- Read this manual carefully before installation, operation and adjustment. And please keep it well.
- Comply with safety operation rules, product shall not be overloading, and to ensure that the product safety protection equipment is complete and reliable, eliminates unsafe factors on time.
- Tighten bolts and nuts regularly to avoid failure caused by loose, sliding, vibration and falling off.
- This product is not allowed to operate when it's with fault. If abnormal phenomenon or abnormal sound is found, it must be stopped for inspection immediately, and work can be resumed after troubleshooting.
- The operator must be familiar with the precautions described in each section of the manual to ensure the safety of person and product.
- If the casters are damaged, please replace them before use.
- After every treatment, please only use neutral solution to clear the dust or stains on the product.
- Put the product to a proper location, according to the overall layout, the daylighting or artificial lightings and operation condition of the working environment. And keep it in the clean, dry and cool environment.

4. Installation


1. Open the packing carton to check if the product is intact and whether the accessories or spares are complete. Please contact distributor or us if there's any problem.
2. Installation

This product graphic is only for display, the specific product structure please refer to the actual product.


1025 / 1025A / 1025B / 1025C · Installation procedure

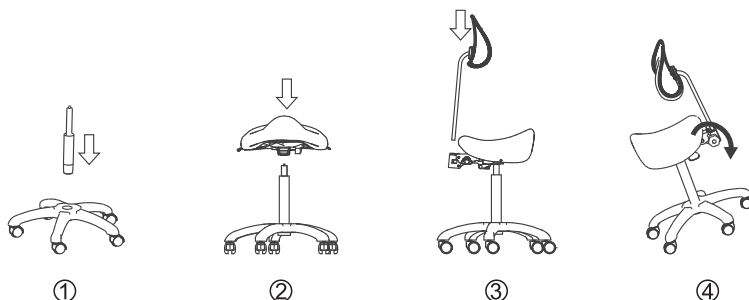


1025 / 1025A / 1025B / 1025C • Operation Instructions

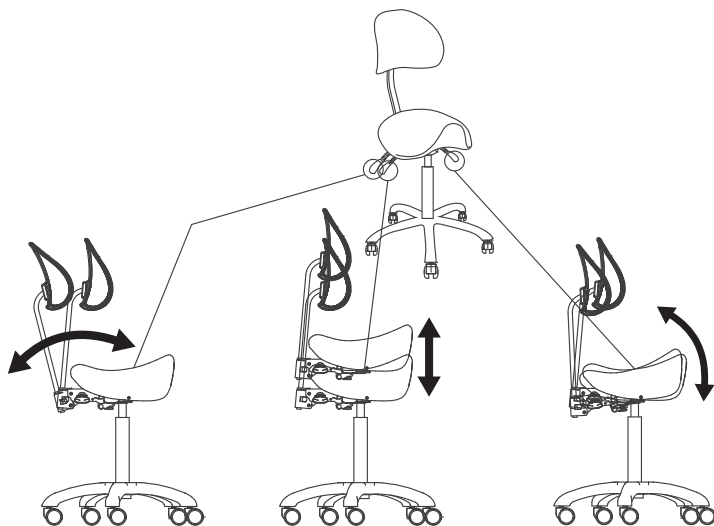


1026AB2 • Installation procedure

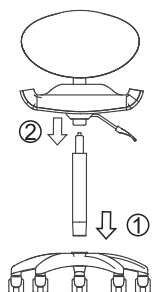

1026CB2 • Installation procedure

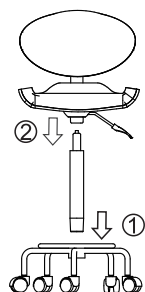

1026AB5 • Installation procedure

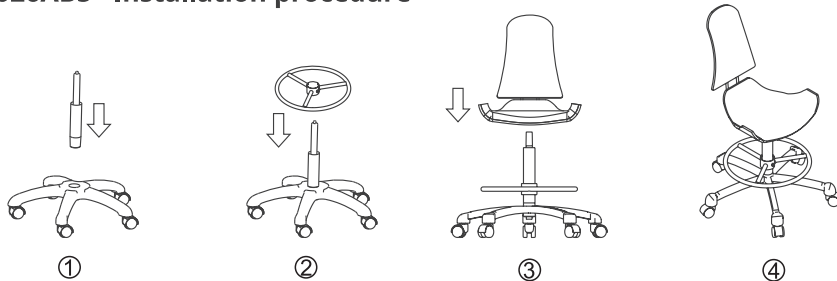


5. Technical Parameters


This product illustrations in this manual are based on the configuration parameters of standard gas lift. If it is equipped with another optional gas lift, the parameters are subject to the actual product.

Lifting Rated Load	175 kg (1025 / 1025A / 1025B / 1025C) 150 kg (1026AB2 / 1026CB2 / 1026AB5)
Package Size	59×30×60 cm (1025 / 1025A / 1025B / 1025C) 55×48×47 cm (1026AB2) 55×48×45 cm (1026CB2) 59×46×54 cm (1026AB5)


1025

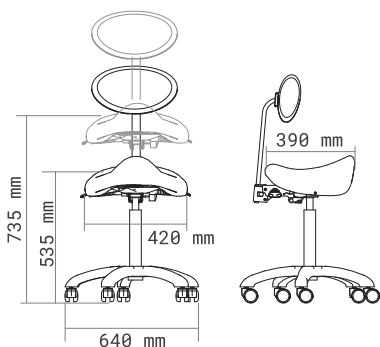


1025A

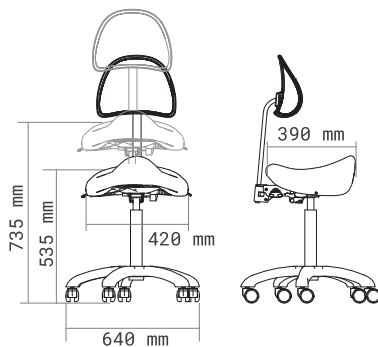


1025B

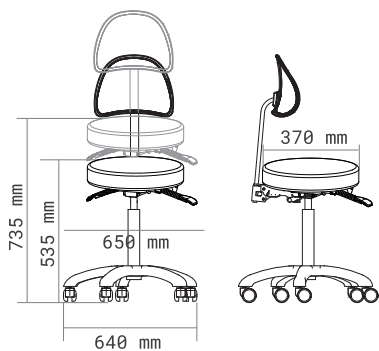


1025C

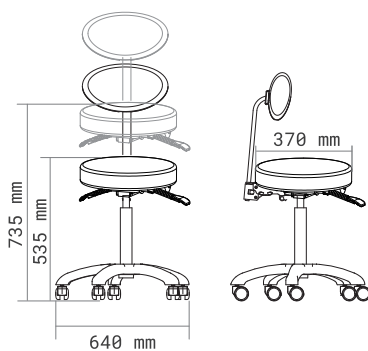


1026AB2

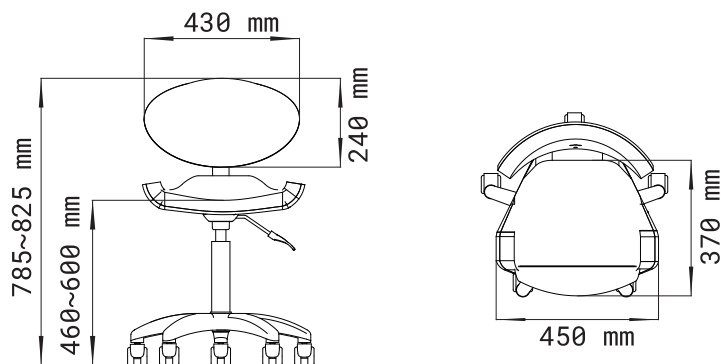

1026CB2

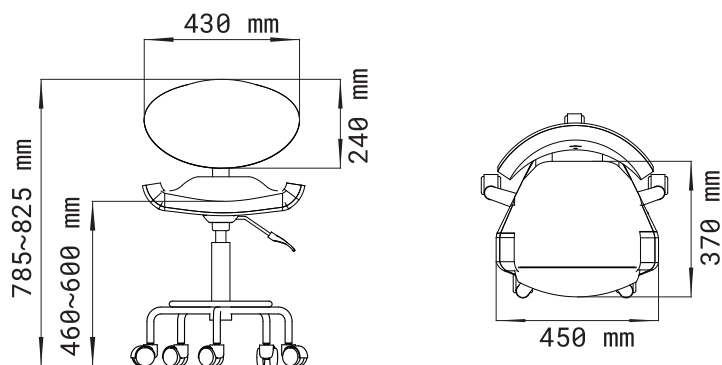

1026AB5

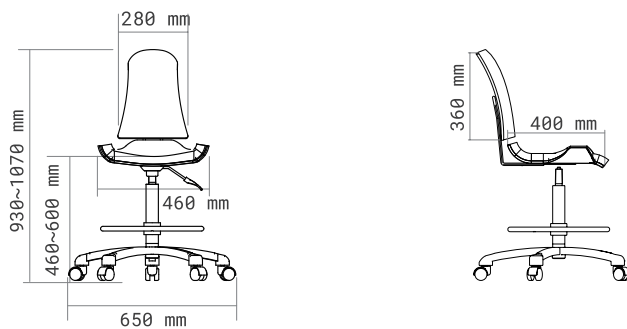

6. Maintenance

1. After every treatment, please only use neutral solution to clear the the dust, stains on leather, metal part or other parts. Disinfecting solution like alcohol, acid or alkaline solution are forbidden. This is to prevent the lifetime of the components from being shortened.
2. Please lubricate the moving parts regularly (once a year)

7. Troubleshooting

The Fault Phenomenon	Reason	Solution
Not working properly	1. Can not raise or lower the stool.	1. Replace the gas lift.
	2. Casters can not rotate or move.	2. Replace the caster.

If problem is still not solved after above operations,
please contact maintenance center or supplier to solve it.

▲ Please check the product specification before use. This manual is for reference only. Specifications are subject to change without prior notice.