


Instructions Manual

Beautyinstrument

WKB019

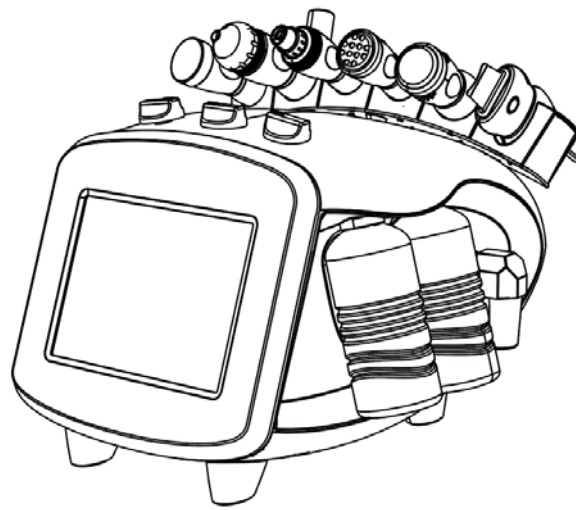

VERSION 1.26

CORRECT USE OF THE DEVICE

- This product has been designed to be used by beauty professionals at beauty salons and should not be used for other purposes.
- Please note that any misuse of the device entails the loss of the guarantee.
- Do not make any modification to the device without the authorisation
- we shall not be held liable for any material or personal damage due to the misuse of the product.

CONTRAINDICATIONS

- This product should not be used on:
 - People with metal implants in their body.
 - People with pacemakers or heart problems.
 - Pregnant women.
 - People with sensitive skin, lesions, inflammations, or any skin problem or disease.
- With people undergoing medical treatment or a diagnostic process, the product must be used with medical authorisation.

WARNINGS

When using the ultrasonic function:

- It is prohibited to leave the probe in the output state without cosmetic operation, as this will cause the probe to overheat and rapidly reduce its service life.
- Do not attempt to separate the stainless steel probe from the handle, as frequent rotation of the stainless steel probe with the handle may cause damage to the probe.
- It is strictly forbidden to put the probe under water for cleaning, as this may cause damage to the probe.

It is strictly prohibited to use probes with broken surfaces or serious scratches, and if there is such a phenomenon the accessories must be replaced.

When using Skin scrubber function:

- It is strictly prohibited to leave the scrubber in the output state for a long period of time without carrying out aesthetic operations, otherwise it will cause the scrubber head to overheat and rapidly reduce its service life and risk scalding the skin!
- Do not splash the host with water or other corrosive liquids when using it.
- Do not use a scrubber whose stainless steel surface has been deformed and chipped to avoid skin damage.
- Unauthorized opening of the housing for maintenance is prohibited. If there is any abnormality,
 - please contact your dealer and have it checked by a professional repairer.
- It is strictly prohibited to put the probe in water for cleaning, and it is strictly prohibited to use alcohol to wipe the parts of the probe other than the metal parts, which may cause damage to the probe.


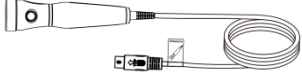

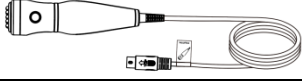

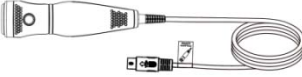

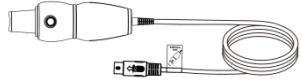



ATTENTION

- After the first use and after each use, check the appliance and the power cord for damage, always plug the appliance into a correctly fitted socket, the power socket must be well earthed and easily accessible so that the appliance can be quickly unplugged at all times. The local supply voltage must correspond to the technical specifications of the appliance. Pull out the plug directly, not the power cord, and never touch the equipment or power cord

when your hands are wet!


- At the end of the workday, disconnect the unit and unplug the power cord from the power source.
- When is working, pay attention to the wiring between the accessory and the host computer and don't overstretch it.
- If you find any abnormalities in the host during the working process, please turn off the machine immediately and unplug the power supply, please contact your nearby dealer for inspection and maintenance or ask a professional maintenance personnel to check, it is prohibited to open the casing for maintenance without authorization.
- Prohibit placement and use in bathrooms or damp areas.
- Prohibit placement with flammable and explosive objects.
- This equipment and accessories should be kept out of the reach of children.
- Do not use this product outdoors or in areas that may be affected by moisture or inclement weather.
- Unplug the unit before starting to clean or perform any type of maintenance.

Packing List


Name	Graphic	QTY	Name	Graphic	QTY
Host Machine		1 Round	Ultrasonic probe		1 Piece
Power Cable		1 Piece	Poration probe		1 Piece
Aqua cleaning probe (non-removable)		1 Piece	Cold and hot hammer probe		1 Piece
SPRAY WATER OXYGEN probe (non-removable)		1 Piece	Skin scrubber probe		1 Piece
Suction head		1 Set	Instruction Manual		1 Copy
water bottles ABCD		4 Piece			

Introduction to the annexes

Ultrasonic

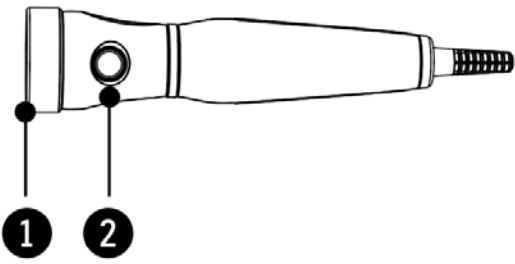


1.Metal probe
Skin scrubber
2.Handle switch

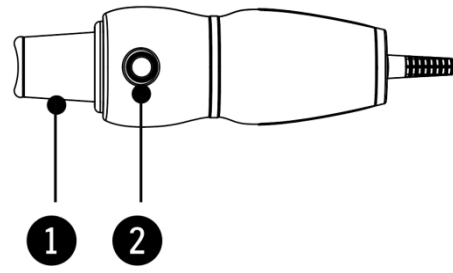


1. scrubber head
2.Handle switch

Spray water oxygen

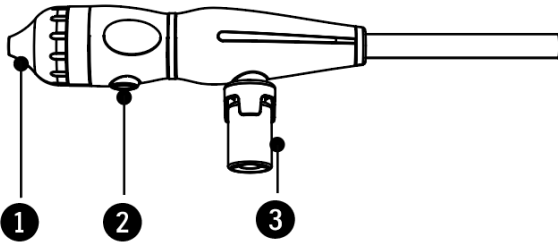


1. Spray Hole
2. Handle switch
3. SPRAY WATER OXYGEN with water bottles

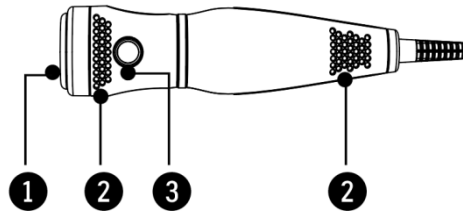


1. Cold and Hot hammer Probe
2. Heat sink
3. Handle switch

Cold and Hot hammer

Aqua cleaning

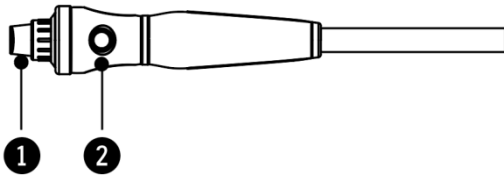


1. Suction head
2. Handle switch

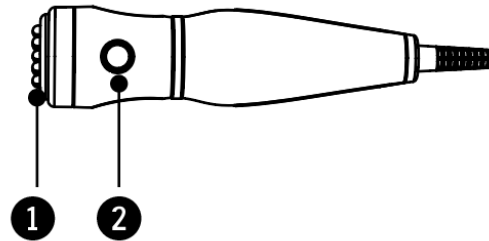

1. Metal contact
2.Handle switch


**Poration
Host Introduction**

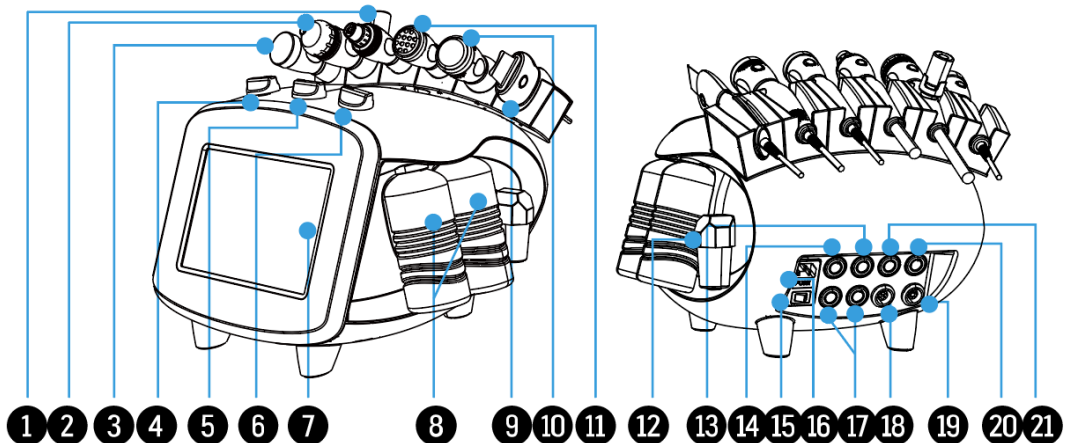

1. Aqua cleaning probe
3. Ultrasonic probe
5. Selection of water bottle knob
7. Control panel
9. Skin scrubber probe
11. Poration probe
13. Cold and hot hammer Socket
15. Power switch
17. Reserve a socket
19. Spray water oxygen output interface
21. Poration sockets
2. SPRAY WATER OXYGEN probe
4. Water volume adjustment knob
6. Suction Adjustment Knob
8. ABCD water bottles
10. Cold and hot hammer probe
12. Filter
14. Skin scrubber Socket
16. Electric socket
18. Aqua cleaning output interface
20. Ultrasonic sockets

Preparation of the instrument before use:

1. Connect the power cord of the host.

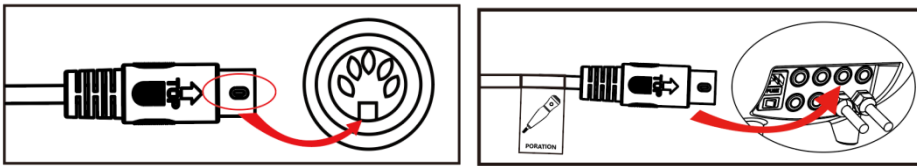

2. Plug the probes into the corresponding output ports on the back of the host respectively. Note that the label on the flexible cable must correspond one by one with the output port on the back of the host, and the arrow on the plug should correspond to the corresponding position of the output socket of the host.
3. Fill the A,B,C water bottles with appropriate amount of mineral water, tap water, cleaning solution or serum respectively, and then screw them back to the corresponding position on the body. **Note:** D bottle is a waste water bottle, used to collect used waste water, can not be filled with other water.
4. Please install a water bottle on the Spraywater oxygen handle and bubble pen before



Function Selection Screen



1. Select Ultrasonic function;
2. Select spray water oxygen function;
3. Select Aqua cleaning function;


4. Select Poration function;
5. Select Cold and hot hammer function;
6. Select Skin scrubber function;

Function

Aqua cleaning

Effect

- Deep Cleansing: Adsorbs blackheads, oil and keratin debris from pores through vacuum negative pressure.
- Water-oxygen nourishment: with special solution (containing fruit acids, salicylic acid, etc.) to soften the keratin, synchronized with the injection of hydrating essence.
- Improve skin texture: reduces closed-head acne and blackheads, balances water and oil, and improves skin clarity.

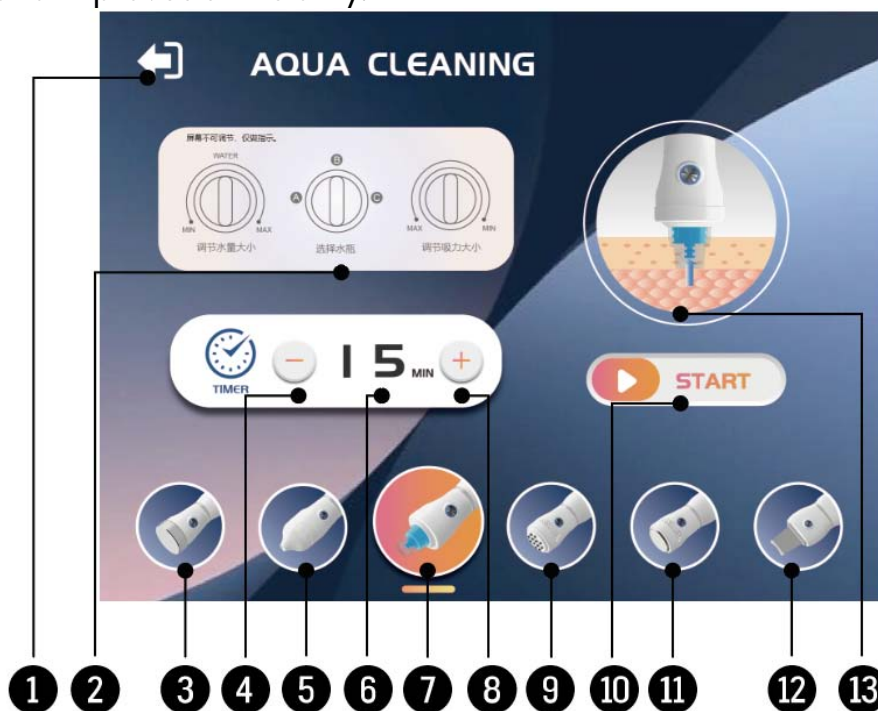

1. Return key: return to the function selection interface;
2. On-screen instructions: indicate the function and operation of the three knobs;
3. Ultrasonic selector key: go to ultrasonic function;
4. Time minus key: reduces the operating time output;
5. Spray water oxygen selection key: go to spray water oxygen function;
6. Time display window: displays the working time;
7. Aqua cleaning selection key: shows current selection status;
8. Time increment key: increase operation time;
9. Poration selector key: go to poration function;
10. Start button: start working;
11. Cold and hot hammer selector key: go to cold and hot hammer function;
12. Skin scrubber selector key: go to skin scrubber function;
13. Function display window: displays the current function schematic screen;

First fit the handle with a Suction head,



1.Frosted head 2.Hard plastic head 3.Silicone head

Comes with three types of Suction heads:

- The scrub head is suitable for getting rid of exfoliating dead skin, women's skin is thinner, so only do the T-zone, only once, men can operate on the whole face according to the situation.
- The hard plastic head is suitable for regular skin use, different models target different areas for full face cleansing, and if you need to do nutrient solution introduction, use spot plucking.
- The silicone tip is suitable for sensitive skin, non-irritating and more comfortable.

Functional adjustment



In the function selection interface, press " " or click on the shortcut toggle button below in the other function screens.




Select the Aqua cleaning feature

by default:




" **TIMER** " The time display shows "15", in which you can adjust the operating time of

the function. Select the suitable working time by " + " and " - "; Adjustment range: "1 to 30" minutes.

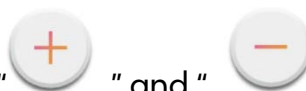

After adjusting the relevant settings, it is possible to adjust the settings by pressing the

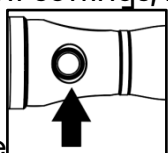

start button on the handle

Or just press or to




or



a

grooming operation.

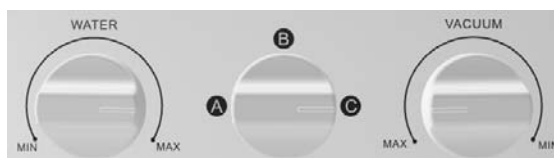


Three knobs on the body adjust the different functions separately as follows:


adjust the different functions separately as follows:



"WATER" knob: Adjust the size of water output, MIN means minimum, MAX means maximum.

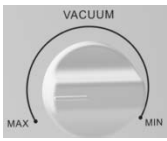

"VACUUM" knob: Adjust the suction power, MIN means minimum, MAX means maximum.

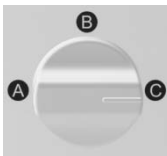

Water bottle selector knob: select A bottle, B bottle, C bottle (corresponds to



the water bottle on the body)



There are also specific instructions for use on the screen.

How to use

1. Preliminary

- **CLEAN SKIN:** After removing makeup and cleansing your face, use the hot spray function or hot towel treatment for 3-5 minutes to open your pores.
- **Preparation solution (for reference only)**
 - **Bottle A (cleanser):** contains lactic acid and salicylic acid to soften cuticles (can be diluted for sensitive skin).
 - **Bottle B (serum):** Contains hyaluronic acid, B5, etc., hydrating and repairing.

2. Operation process

Select 1-2 speeds, fit the small bubble probe to the skin, and slide it in an "inside-out, bottom-up" motion. Because the skin's negative pressure tolerance time is about 0.3 seconds/mm², it is recommended to move 1-2cm per second. key areas: nose, forehead, chin adsorption in a circle (each stay ≤ 3 seconds), avoiding the sensitive areas around the eyes and cheeks. Observe the adsorbent: yellow oil or black dirt visible in the transparent tube is the normal effect.

- Frequency of use: oily/combo skin: once a week, dry skin: once every 2 weeks, sensitive skin: once a month or as directed by a doctor.

3. Aftercare

- **Instrument cleaning:** Immediately rinse the pipette tip and tubing to prevent clogging with solution residue. Disinfect the probe with alcohol cotton pads, air dry and store.
- **Skin Care:** Avoid using exfoliating products or strong whitening serums for 3 days.
- **Enhanced sun protection** (skin is more fragile after cleansing), physical sun protection is recommended.

CAVEAT

- **Contraindicated for:** severely sensitive skin (redness and swelling stage), rosacea, herpes and other skin diseases. During pregnancy and breastfeeding (avoid penetration of acid solutions).
- **Prohibited areas:** around the eyes, cheekbones and other skin weakness, facial filler areas (hyaluronic acid / thread sculpture areas). If the solution accidentally enters the eyes, immediately flush the eyes with saline, apply erythromycin ophthalmic ointment and rest away from light for 24 hours.
- Suction control: "slight redness of the skin but no pain" as the safety threshold, prohibited for the pursuit of the effect of forcibly adjusting the high grade. If purplish-red bruises appear, stop using it immediately and apply cold compresses.
- Solution selection: avoid mixing their own liquid of unknown composition to prevent corrosion of the instrument or skin irritation. Sensitive skin can use pure water or mild chamomile solution instead of acid-based cleaning solution.
- If in the process of use, the instrument stops working to send out "dee,dee" alarm



sound, the handle on the switch key light off, the panel on the orange color, **that the waste water bottle is full!** At this time, you need to unscrew the water bottle D, clean up the waste water inside and load it back, and make sure the waste water bottle is tightened! Wait a few seconds to resume use (the alarm will turn off and the on/off light on the handle will come on).

- The suction head should be kept clean, clean and sterilize the suction head after each use. Note: If you use alcohol to sterilize only wipe, the metal part of the suction head, do not wipe other parts of the probe, otherwise it will damage the suction head.
- Nursing time is generally no more than 15 minutes, the care part of the area is small to reduce the appropriate time of care, too long will make the skin fatigue and malabsorption; probe should not stay in the skin for a long time in a certain position, so as not to damage the skin tissue.
- During the care process, you should avoid doing beauty treatment for a long time in the state of maximum energy output, this state will not only affect the absorption of the skin, and even damage the skin tissue.

Upkeep

- Pipe maintenance: Flush the pipe with white vinegar + water (1:3) every month to dissolve scale and grease residue. Empty the solution when not in use for a long time and keep it dry.
- Probe Replacement: Silicone tips are recommended to be replaced every 3 months.


ULTRASONIC

Effect

- Soften and remove fat to cure acne: there are various forms of acne, the more common are suppurative and acne, ultrasound can impact the lymph nodes, accelerate the accumulation of blood and lymph circulation dissipation, inflammatory cells in the role of ultrasound to change the shape of the ultrasound, coupled with ultrasound will be anti-inflammatory drugs into the congestion of swollen acne to improve the phenomenon of the skin on the nodule gradually soften.
- Elimination of freckles and lightening of pigmentation: ultrasonic waves can destroy the inner membrane of pigmented cells, prevent it from forming and decompose the

pigment, so that with the metabolism and make the spots fade to disappear; ultrasonic waves can enhance the absorption of the care products (e.g., traditional Chinese medicinal creams, high unit of Vitamin C, pigmentation creams, etc.), and the effect of elimination of pigmentation is remarkable.

- Resolving blockages and knots: the use of ultrasound mechanical massage effect, on the one hand, regulates the subcutaneous cells and enhance the permeability of cell membranes, on the other hand, it strengthens the blood circulation and metabolism; with the appropriate blood circulation and blockage of the effect of the care products is significant.
- Eliminate eye bags and dark circles, due to the mechanical massage effect of ultrasonic beauty instrument can accelerate blood and lymphatic circulation, coupled with the role of heat, can refine and decomposition of fat, accelerate the absorption of the subcutaneous, so that the excessive accumulation of water and fat dissipation, eye bags and dark circles will disappear. (When doing eye care with ultrasonic beauty instrument, the probe should be massaged along the direction of the skin texture, and each eye bag care should not be more than 5 minutes, the intensity should be gentle, the probe can not be directed at the direction of the eyeballs.)

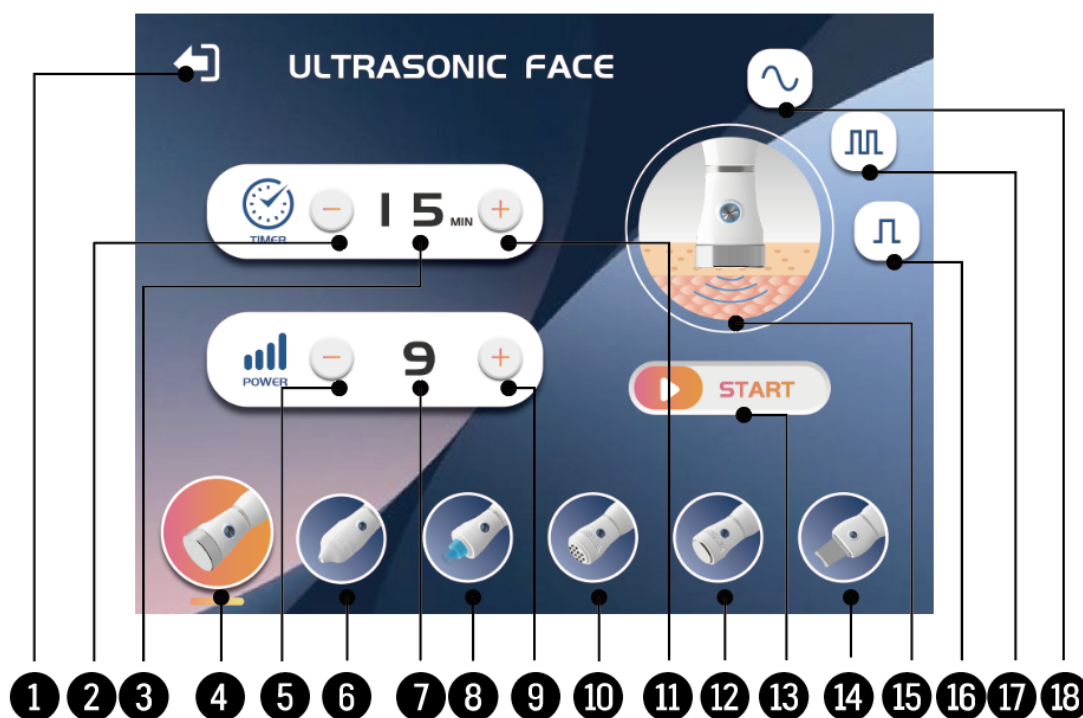

1. Return key: return to the function selection interface;
2. Time reduction key: reduces the operating time output;
3. Time display window: displays the working time;
4. Currently in use: ultrasonic function;
5. Energy reduction key: reduces energy output;
6. Spray water oxygen selection key: go to spray water oxygen function;
7. Energy display window: displays the intensity of energy output;
8. Aqua cleaning selector key: go to aqua cleaning function;
9. Energy booster: increase energy output;
10. Poration selector key: go to poration function;
11. Time increase key: increase operation time;
12. Cold and hot hammer selector key: go to cold and hot hammer function;

- 13. Start button: start or pause work;
- 14. Skin scrubber selector key: go to skin scrubber function;
- 15. Function display window: displays the current function schematic screen;
- 16. Wide pulse wave selection key: selects wide pulse wave output;
- 17. Mixed wave selection key: selects the mixed wave output;
- 18. Continuous wave selection key: selects the continuous wave output;

Functional adjustment



In the function selection interface, press " " or click the function shortcut switch below



in the other function interface
Selects the ultrasonic function, by default:




"The energy display shows "9"



"The time display shows "15" minutes. In this state, you can adjust the working time of the function and the amount of energy output.

Energy Adjustment: Select the appropriate output energy as required in

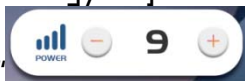


"through " + "and " - "; Adjustment range "1~9 " gear.

Time Adjustment: Select the appropriate working time as required in



"through " + "and " - "; Adjustment range " "1 to 30 " "minutes.

After adjusting the relevant settings, it is possible to adjust the settings by pressing the

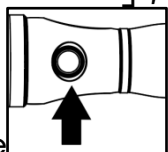

start button on the handle

Or just press or to  or  a grooming operation.

Waveform output state selection: "  "Continuous wave output state; "  "Mixed

wave output state; "  "Wide pulse wave output state.

How to use

1. Cleanse facial skin with appropriate cleansing lotion before use.
2. Wipe off the cleansing lotion.
3. Gently pat on all areas of the face with the appropriate astringent water. Gently wipe off the astringent water for complete disinfection and optimal protection of the skin.
4. Analyze the nature of the skin and select the appropriate ointment to spread evenly on

the face, subject to the flexible rotation of the probe during operation.

5. Turn on the ultrasonic function, adjust the energy size, generally do face can be adjusted to the strongest 9, do the eye to the middle position 5 can be.
6. Direct contact between the probe and the skin: the probe moves uniformly in the treatment area, or moves slowly in a straight line back and forth in a slow circle, moving at a speed of 0.5----2.5CM per second.
7. Operation time, usually about 15 minutes is sufficient.

Selecting waveforms as needed.



Continuous wave suitable for normal skin,



Mixed Wave for Dry, Oily Skin,



Wide Pulse Wave is suitable for skin with a thin stratum corneum that is prone to allergies.

8. Turn off the instrument when the time is up or the operation is complete.


CAVEAT

- The probe should be kept clean and cleaned and sterilized with alcohol or other disinfectant tool after each use.
- When using ultrasound, never point the probe at the eye.
- When the surface of the skin is not enough care serum has moisturized the skin, it should be replenished.
- Nursing time is generally no more than 15 minutes, the care part of the area is small to reduce the appropriate time of care, too long will make the skin fatigue and malabsorption; probe should not stay in the skin for a long time in a certain position, so as not to damage the skin tissue.
- During the care process, you should avoid doing beauty treatment for a long time in the state of maximum energy output, this state will not only affect the absorption of the skin, and even damage the skin tissue.

SPRAY WATER OXYGEN

Effect

- **High-pressure penetration:**The nutrient solution is atomized into nano-sized particles ($\leq 0.3\mu\text{m}$) by 0.3-0.5MPa pressure, which directly penetrates the stratum corneum.
- **Instant Oxygen Replenishment:** High-speed airflow carries oxygen molecules into the skin to improve cellular hypoxia.
- **Repair Barrier:** Promotes collagen synthesis, strengthens the water locking capacity of the stratum corneum and relieves sensitivity and redness.
- **Enhanced Skin Care:** Increase the absorption rate of active ingredients by 3-5 times (especially suitable for large molecule hyaluronic acid and peptides).

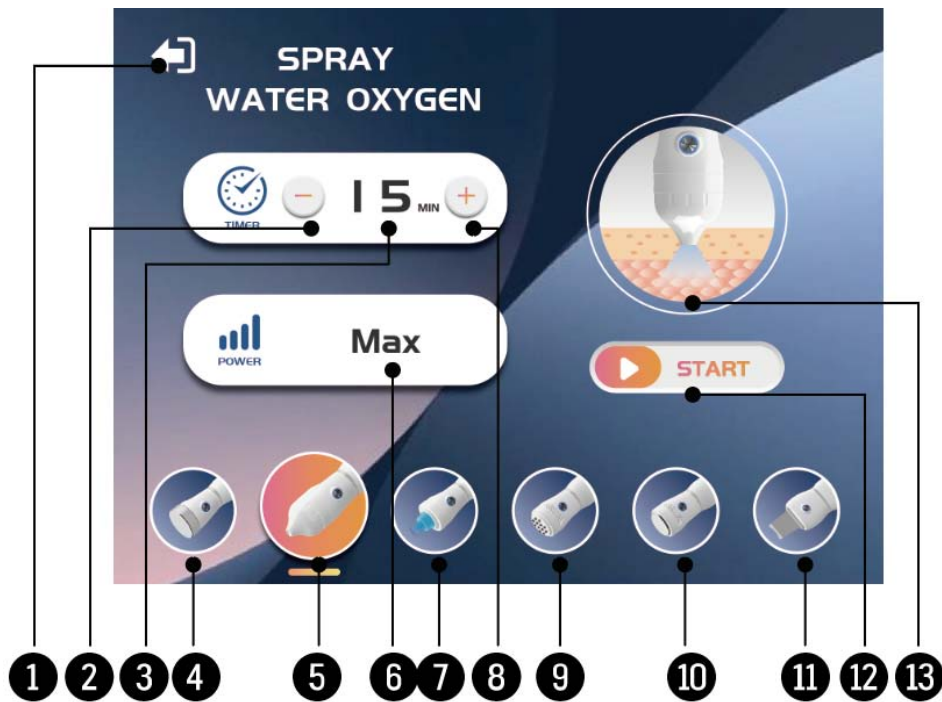

1. Return key: return to the function selection interface;
2. Time reduction key: reduces the operating time output;
3. Time display window: displays the working time;
4. Ultrasonic selector key: go to ultrasonic function;
5. Currently in use: spray water oxygen function;
6. Energy display window: displays the maximum value of energy;
7. Aqua cleaning selector key: go to aqua cleaning function;
8. Time increment key: increase operation time;
9. Poration selector key: go to poration function;
10. Cold and hot hammer selector key: go to cold and hot hammer function;
11. Skin scrubber selector key: go to skin scrubber function;
12. Start button: start working;
13. Function display window: displays the current function schematic screen;

Functional adjustment



In the function selection interface, press " " or click on the shortcut toggle button below in



the other function screens. Select the hyperbaric oxygen injection function by default:



" POWER " "Max" appears on the energy display.



"TIMER" "The time display shows "15" minutes, In

this state, you can adjust the working time of the function output.




The Energy Adjustment Window "is fixed to the maximum value and cannot adjust the energy level.

Time Adjustment: Select the appropriate working time as required in



through " "and " "; Adjustment range "1 to 30 " "minutes.

After adjusting the relevant settings, it is possible to adjust the settings by pressing the start

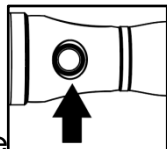

button on the handle Or just press or to a grooming operation.

How to use

Preliminary


- Skin cleansing
 - After rinsing the amino acid cleanser, rinse with ionized/pure water (to avoid clogging the nozzle with tap water calcium and magnesium ions).
 - Use the hot spray function for 2 minutes to open the cuticle gap (omit this step for sensitive skin).
- Solution preparation (for reference only)
 - Basal fluid: saline/sterile distilled water (70%).
 - Efficacy essence: water-based essence (e.g. hyaluronic acid original solution, B5, lyophilized powder compound solution, 30%).
 - Prohibited additions: oils, lotions, highly concentrated acids (pH<3.5), alcohol (>5%).
- Instrument assembly:
 - Pour the mixture into the water bottle (do not exceed the MAX line) and screw on the gun.
 - Turn on the machine and spray air for 2 seconds (nozzle facing down) to check the atomization uniformity.

How to use (keep the gun perpendicular to the skin the whole time!)

Shore	Gap	Travel speed	Points to note	Length of time
Eyebrows	10-12cm	1cm/sec	Move up from the center of the brow to the hairline, 3 round trips in parallel lines	1 minute
Cheeks	8-10cm	0.5cm/sec	Spiral circles from the side of the nose to the front of the ear, focusing on the reddened areas	1.5 points per side
Ala nasi	12-15cm	Whip	Avoid nostrils! Tilt at 45° to gently sweep oil from the T-zone	20 seconds
Jawline	5-8cm	2cm/sec	Slide from the center of the chin along the edge of the bone behind the ear to lift and	40 seconds

Shore	Gap	Travel speed	Points to note	Length of time
			tighten	

Pivot

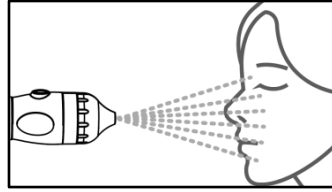

- The SPRAY WATER OXYGEN gun is always **90° perpendicular** to the skin; tilting it will cause the liquid to splash.
- Press and hold the gun for **continuous spray**, tap for **pulsed spray** (pulsed recommended for sensitive skin).
- **Use ≤ 5ml of liquid** for a single full face application (overdose can easily lead to hydrated dermatitis).

CAVEAT


- Prohibition on use
 - Facial herpes/infections, open wounds, rosacea outbreaks.
 - Patients with severe asthma (high pressure airflow may induce wheezing).
- Use with caution
 - Avoid adding potent extracts during pregnancy (use only pure water) and use with caution for scarring.
- Safe operation
 - Distance < 5cm → liquid high-pressure impact damages the dermis.
 - Distance > 15cm → atomization fails and liquid drops are wasted.

Prohibited ingredient

Prohibited ingredient	Exposures	Alternative
Essential oils/oil-based products	Clogged nozzles + risk of lipid pneumonia	Water-based extract
Granular frosting (e.g. diamond dust)	Wear metal nozzles	Liquid Acid Exfoliation
High concentration of VC (>15%)	Oxidation crystallization corrosion instruments	VC derivatives up to 5%

Upkeep

- Daily Maintenance
 - Immediately disassemble the gun after each use of the function, clean the liquid in the bottle rinse with distilled water otherwise leakage will occur as shown in the figure below.

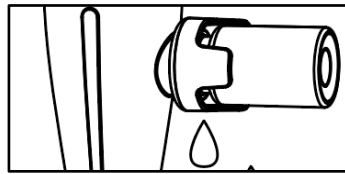


- The nozzle is wiped in one direction with a cotton swab dipped in alcohol (do not poke it in the opposite direction).
- After use empty spray distilled water for 10 seconds to clean the line, invert to dry.
- Weekly deep cleaning
Disassemble the nozzle by soaking it in a white vinegar: water (1:4) solution for 15 minutes and brushing it lightly with a soft bristle brush.
- Consumables replacement
 - Nano-filter: change every 3 months (immediately when atomized particles become coarse).
 - Seal ring: Replacement every 6 months (to prevent leakage leading to insufficient pressure).

PORATION

Effect

- Temporarily open the skin channel:
The microcurrent acts on the skin cell membranes to temporarily change the structure of the lipid bilayer of the cell membranes to form hydrophilic micropores (nanoscale channels). These channels allow macromolecules and water-soluble ingredients (e.g. vitamin C, hyaluronic acid, peptides, etc.), which would otherwise be difficult to penetrate the stratum corneum, to enter directly into the deeper layers of the skin.
- Charge-driven component migration:
Utilizing the principle that "like repels like and opposite attracts like":
Positive Ion Mode (+): drives the penetration of positively charged ingredients (e.g. most vitamin C derivatives, minerals) into the skin.
Negative Ion Mode (-): drives negatively charged ingredients (e.g. hyaluronic acid, salicylic acid) deeper into the skin.
- The current actively "pushes" the charged components to the target cortex, far more efficiently than passive diffusion.

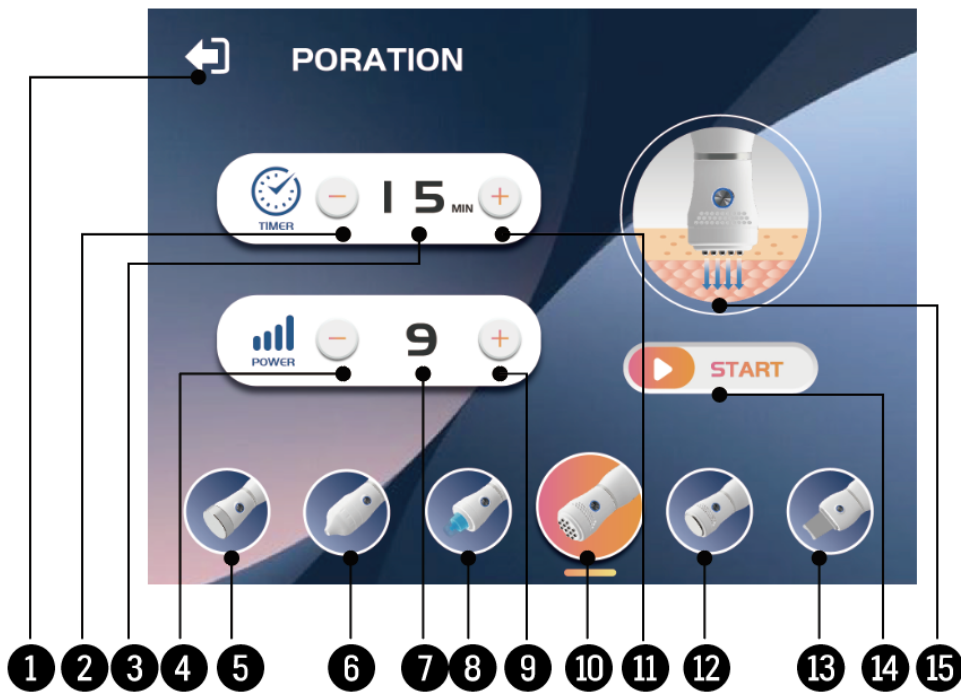

1. Return key: return to the function selection interface;
2. Time reduction key: reduces the operating time output;
3. Time display window: displays the working time;
4. Energy minus key: reduces energy output;
5. Ultrasonic selector key: go to ultrasonic function;
6. Spray water oxygen selection key: go to spray water oxygen function;
7. Energy display window: displays the intensity of energy output;
8. Aqua cleaning selector key: go to aqua cleaning function;
9. Energy boost: increase energy output;
10. Poration selection key: displays current selection status;
11. Time increment key: increase operation time;
12. Cold and hot hammer selector key: go to cold and hot hammer function;
13. Skin scrubber selector key: go to skin scrubber function;
14. Start button: start working;
15. Function display window: displays the current function schematic screen;

Functional adjustment:



In the function selection interface, press " " or click on the shortcut toggle button below in the other function screens.



default:

Select the poration function, by



POWER



TIMER

"The energy display shows "9"." "The time display shows "15" minutes, In this state, you can adjust the working time of the function and the amount of energy output.

Energy Adjustment: Select the appropriate output energy as required in




"through " + "and " - "; Adjustment range "1~9 " gear.

Time Adjustment: Select the appropriate working time as required in



"through " + "and " - "; Adjustment range " "1 to 30 " "minutes.

After adjusting the relevant settings, it is possible to adjust the settings by pressing the

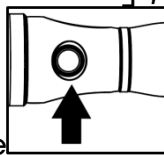

start button on the handle

Or just press or to



a

grooming operation.

How to use

1. Cleanse your face thoroughly:

Cleanse your face with a gentle cleanser to ensure that your skin is free of oil, dirt and makeup residue. Clean skin is the foundation for effective introduction. Gently pat your face dry with a clean towel.

2. Choose the right serum:

- A **water-based, oil-free, alcohol-free** serum must be used (preferably a product specifically designed for iontophoresis).
- Serums need to contain ionized (charged) active ingredients (e.g. vitamin C, hyaluronic acid, peptides, niacinamide, etc.) so that the microcurrent can drive them to penetrate.
- Avoid: oily serums, creams, gels (unless explicitly stated to be used for electroporation), products containing alcohol (will sting and may damage the instrument).

3. Apply a sufficient amount of serum:

- Apply an even layer of a sufficient amount of moist serum to the area to be worked on (e.g., full face, neck). Serum is a conductor of electric current. Too little amount will result in poor conduction of electric current, tingling sensation or poor results.
- Ensure that the surface of the skin remains moist and feel free to replenish the serum if it feels dry during the operation.

4. Turn on the instrument and select the mode:


Selection of "PORATION" function

5. Set the intensity:

- Must start in the lowest gear!
- Fit the device gently and completely onto the serum-applied skin.
- Slowly turn up the intensity level until you can feel a very slight, comfortable tingling or warming sensation. Never aim for a strong tingling sensation! If you feel discomfort or pain, immediately lower the gear or stop using it.

6. Manipulation:

- Move slowly: Slide the instrument over the skin at a very slow speed (about 1-2 centimeters per second). Do not stay in one place for too long (usually no more than 3 seconds) or too fast.
- Tight fit: Ensure that the instrument's metal guide always fits completely and tightly on the moist skin surface. Do not overhang or make only partial contact.
- Unidirectional gliding (recommended): Most recommendations follow the direction of gliding from the inside out and from the bottom up (e.g., forehead from the center to the sides, cheeks from the nose to between the ears, chin to the earlobes, and neck from the bottom up). Follow muscle texture.
- Avoid forbidden areas: Absolutely avoid the eyelids, the week of the orbital bone, the throat (thyroid) area, wounds, inflammation, pimples, moles and scars. Lips are generally not recommended either. Note that gray areas cannot be used.

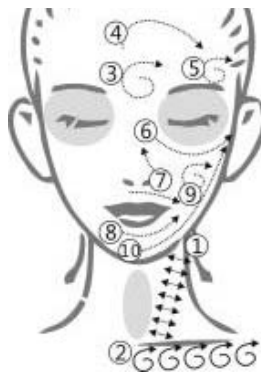

7. Operating time

Usually operate for a few minutes on each area (e.g. left and right cheeks, forehead, chin), keeping the total time for the whole face to about 10-15 minutes. Do not overuse.

8. End operation

- Turn off the instrument when the time is up or the operation is complete.
- Allow the serum to remain on your face for a few minutes to allow it to be fully absorbed. No rinsing is required.
- If you feel any residue or stickiness from the serum, wipe it off with a damp tissue with gentle pressure (don't rub it hard), and then follow up with a skincare step (e.g., applying a lotion or cream to lock in moisture).

9. Cleaning Instruments

Must be cleaned after each use! Thoroughly wipe clean the metal guides and contact surfaces of the instrument with a damp cloth or moistened cotton swab (water or mild soapy water is usually recommended) as directed in the instruction manual to remove residual serum and dirt, and then wipe dry. Avoid getting liquids inside the instrument.

CAVEAT


- **Serum selection is central:** Again, using the wrong serum is not only ineffective, but can also damage the device or irritate the skin. **Try to use the recommended water-based, oil-free, alcohol-free ionizing serum.**

- **The skin must be moist:**Keep the skin covering the operated area covered with enough serum as a conductor throughout. Dryness can lead to stinging and poor results.
- **Intensity is preferred to be low rather than high:**The microcurrent should feel like a very slight comfortable tingling or warming sensation.**Any tingling, burning, or pain is a signal that must be stopped or reduced in intensity immediately.**Don't try to tolerate discomfort.
- **Avoid sensitive and hazardous areas:**Strictly observe the forbidden areas (around the eyes, throat, wounds, etc.). The skin around the eyes is extremely thin and sensitive, and microcurrent may pose a potential risk to the eyeballs.
- **Frequency of use:**It is generally recommended to use 1-3 times per week, adjusted according to the device instructions and your own skin tolerance. Overuse may result in skin irritation or barrier damage.**Do not use it every day.**
- **Contraindicated people:**
 - People with pacemakers or other implanted electronic medical devices. Microcurrents may interfere with the operation of the device.
 - Pregnant and lactating women (insufficient safety data to recommend avoidance).
 - Those with severe skin conditions on the face (e.g., eczema, dermatitis, severe acne, herpes flare-ups), open wounds, infections, unhealed wounds, or large breakouts.
 - People with epilepsy or other neurological disorders (caution or avoidance).
 - Skin cancer or a history of skin cancer.
 - Allergy to metal (guide material).
 - Teeth with metal fillings or dental implants (caution is needed near the operation area to avoid electrical current irritation of the nerve).
- **Use with caution during menstruation:** Some women may have more sensitive skin during menstruation and may consider pausing or reducing the strength.
- **Avoid contact with metal:**During operation, make sure that metal jewelry (e.g., necklaces, earrings) is worn away from the operating area to avoid accidental contact with the instrument's lead generating a current loop.
- **Do not use on mucous membrane areas:**It is forbidden to use it on the lips, inside of the nose, genitals and other mucous membrane areas.
- **Take care of the sun:**Certain imported extracts (e.g. Vitamin C) may increase photosensitivity, making it even more important to increase sun protection after use.
- **Instrument cleaning and maintenance:**Thoroughly clean the guide tip after each use to prevent serum residue from corroding the metal or harboring bacteria. Check the status of the instrument regularly.

COLD AND HOT HAMMER

Effect

- **Hot hammer mode (35-42°C):**Promotes blood circulation, accelerates penetration of skincare products, soothes muscles and improves skin elasticity.
- **Cold hammer mode (15-18°C):**Shrinks pores, calms skin, reduces puffiness and soothes sensitive redness.
- **Alternate:**Stimulates skin metabolism, enhances firmness and improves edematous contours.

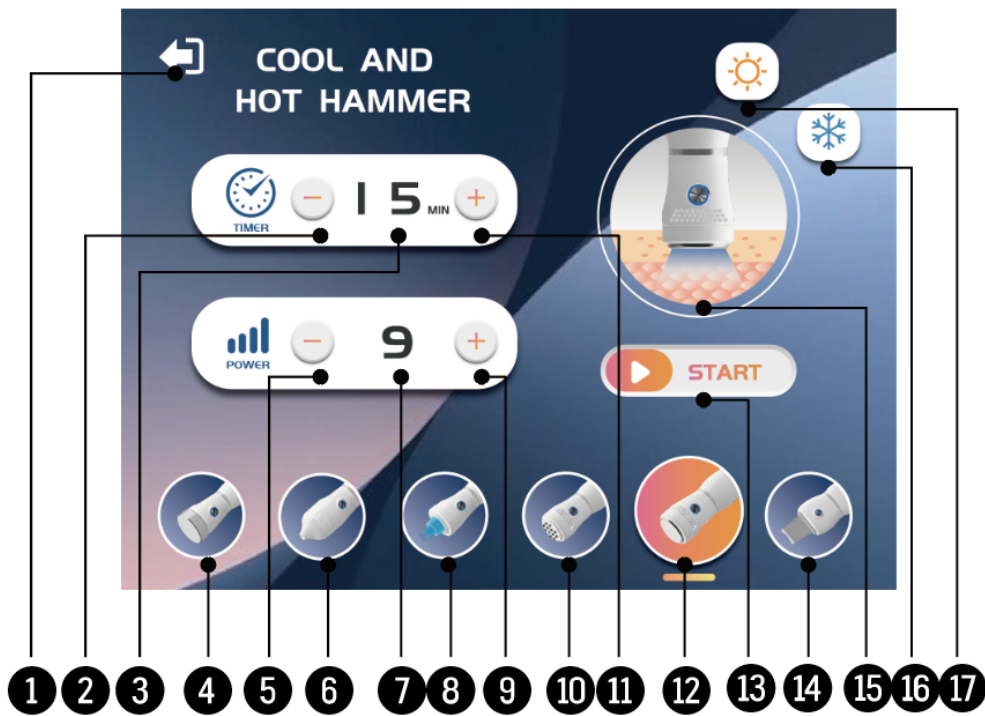

1. Return key: return to the function selection interface;
2. Time minus key: reduce the operating time output.;
3. Time display window: displays the working time;
4. Ultrasonic selector key: go to ultrasonic function;
5. Energy minus key: reduces energy output;
6. Spray water oxygen selection key: go to spray water oxygen function;
7. Energy display window: displays the intensity of energy output;
8. Aqua cleaning selector key: go to aqua cleaning function;
9. Energy boost: increase energy output;
10. Poration selector key: go to poration function;
11. Time increment key: increase operation time output;
12. Cold and hot hammer selection key: displays the current selection status;
13. Start button: start working;
14. Skin scrubber selector key: go to skin scrubber function;
15. Function display window: displays the current function schematic screen;
16. Cold sensing selection key: select cold sensing output;
17. Thermal selection key: selects the thermal output;

Functional adjustment:



In the function selection interface, press " " or click on the shortcut toggle button below in



the other function screens. Select the Cold and hot hammer function, by default:



POWER

"The energy display shows "9"."



TIMER

"The time display shows "15".

In this state, you can adjust the working time of the function and the amount of temp output.

Temp Adjustment: Select the appropriate output temp as required in



"through "  "and"  ";

Select the appropriate temperature (the higher the gear the higher the temperature) Adjustment range "1~9 " gear.


Time Adjustment: Select the appropriate working time as required in



"through "  "and"  ";

Adjustment range "1 to 30 " "minutes.

After adjusting the relevant settings, it is possible to adjust the settings by pressing the start

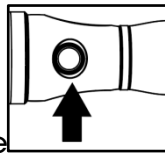

button on the handle

Or just press or to



a grooming operation.

Cold and hot output state selection:"



"is the thermal sensing output state;



"is the cold sensing output state

How to use

1. Preliminary

- CLEAN SKIN: Remove makeup and cleanse the face to ensure no oil residue.
- Apply the medium: Apply gels (for conductive use) or serums (we recommend choosing moisturizers with simple ingredients) as needed.
- Instrument check: The contact tip is clean to avoid impurities scratching the skin.

2. Hot Hammer Mode Operation



- Enabling method: switch to the hot hammer icon
- Maneuver: Lift slowly from the bottom up and from the inside out, stopping for 3-5 seconds per area.
- Focus areas: lines (circular massage), jawline (lifting from chin to behind ears)
- Duration recommendation: 5-8 minutes for a single full face application, 2-3 times per week.

3. Cold hammer mode operation



- Turn on: Switch to Ice Hammer icon.
- Maneuver: Apply light pressure and glide, focusing on enlarged pores (e.g. T-zone) or areas prone to puffiness (under the eyes, cheeks).
- INSTANT CALMING: Can be used alone in cold mode for 3 minutes to cool down after sun exposure or during sensitivity.

4. Cold & Hot Advanced Usage

- Sequence: It is recommended to start with hot and then cold, opening the channel before convergence.
- Example of combination: Hot mode full face massage → apply moisturizing mask → cold

mode to promote absorption.

- WEEKLY CARE: After deep cleansing, introduce serum in hot mode and finish in cold mode.

CAVEAT:

1. Contraindication

- Areas to avoid: eyelids, wounds, acne breakouts, thyroid area (front of neck).
- Contraindicated people: Severely sensitive skin (erythema stage), skin infections, heart patients, pregnant women (avoid the abdomen and lower back).

2. Safety alert

- Temperature control: Hot mode to avoid staying in the same position for a long time (more than 10 seconds may burn).
- Adjustment for sensitive skin: adjust to the lowest gear for the first use, test the ear without discomfort before using the whole face.
- Time management: Do not exceed 15 minutes of total hot and cold time in a single day to prevent overstimulation.

3. Safeguard

CLEANSING: Immediately after use, wipe the contact tip with an alcohol pad to avoid drying out the serum.


4. Matching Suggestions

- Product Compatibility: Avoid using with products containing alcohol, acids (such as salicylic acid) or granular scrubs.
- Follow-up care: After hot or cold stimulation, intensive moisturizing is needed and a soothing cream is recommended.

SKIN SCRUBBER

Effect

The use of ultrasonic transducer technology will be 28,000 times per second of electric shock waves into 28,000 times per second of mechanical vibration waves, the body's muscles have a penetrating effect. Can make the deeper cells under the epidermis to produce excitement, so as long as the use of this instrument with the use of the corresponding potion or serum role, can quickly eliminate black spots wrinkles acne horny layer pores dirt, etc., can be thoroughly clean skin, so that the skin is bright and white, restore elasticity.

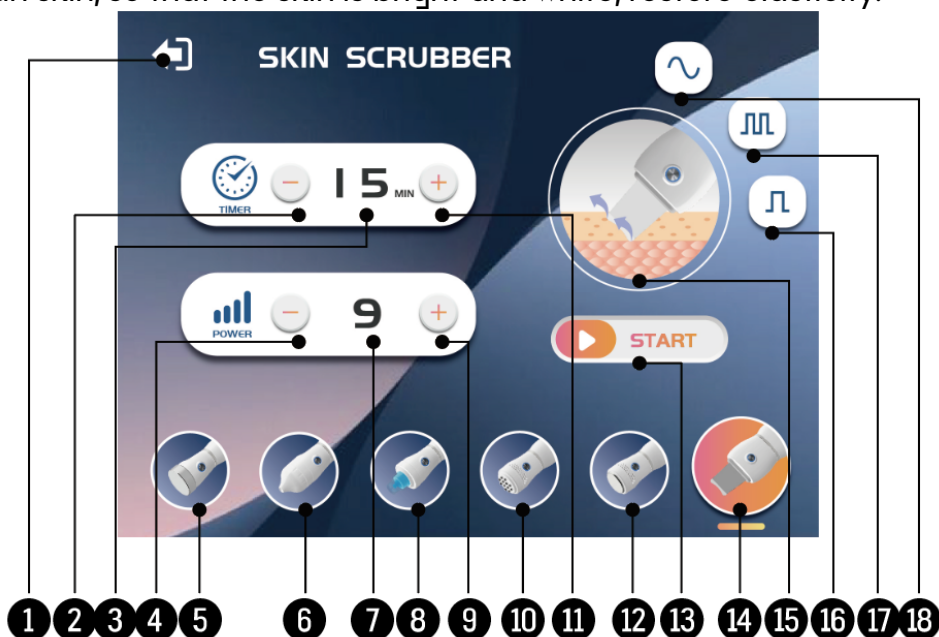

1. Return key: return to the function selection interface;
2. Time minus key: reduces the operating time output;
3. Time display window: displays the working time;
4. Energy minus key: reduces energy output;
5. Ultrasonic selector key: go to ultrasonic function;
6. Spray water oxygen selection key: go to spray water oxygen function;
7. Energy display window: displays the intensity of energy output;
8. Aqua cleaning selector key: go to aqua cleaning function;
9. Energy boost: increase energy output;
10. Poration selector key: go to poration function;
11. Time increment key: increase operation time output;
12. Cold and hot hammer selector key: go to cold and hot hammer function;
13. Start button: start working;
14. Skin scrubber selection key: display the current selection status;
15. Function display window: displays the current function schematic screen;
16. Wide pulse wave selection key: selects wide pulse wave output;
17. Mixed wave selection key: selects the mixed wave output;
18. Continuous wave selection key: selects the continuous wave output;

Functional adjustment:



In the function selection interface, press " " or click on the shortcut toggle button below in



the other function screens. Select the Skin scrubber function, by default:



POWER




TIMER

"The energy display shows "9"." "The time display shows "15".

In this state, you can adjust the working time of the function and the amount of energy output.

Energy Adjustment: Select the appropriate output energy as required in

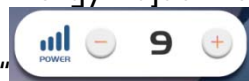


"through " "and " "; Adjustment range "1~9 " gear.



Time Adjustment: Select the appropriate working time as required in



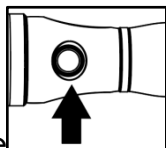


"through " "and " "; Adjustment range " "1 to 30 " "minutes.

After adjusting the relevant settings, it is possible to adjust the settings by pressing the start




button on the handle Or just press or to  or  a grooming operation.

Waveform output state selection:  "Continuous wave output state;  "Mixed wave output state;  "Wide pulse wave output state.

How to use

1. First soften the skin to be cleaned with the hot spray function or water.
2. Adjust the button "ENERGY" to adjust the energy output to the appropriate intensity.
3. Apply skin-cleaning skin care products (e.g., cleanser, etc.) evenly to the surface of the skin to be cleansed, and keep the skin moist throughout the cleansing treatment.
4. Refer to charts A, B, C, and D for methods of presentation.

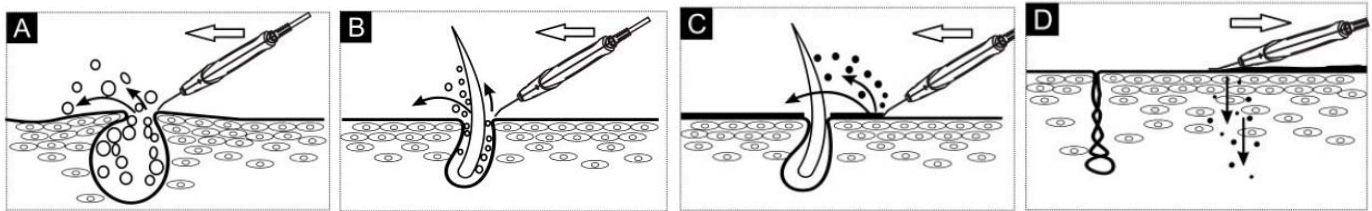

A、Painless Acne Removal (Picture A)

Ultrasound clears the skin in a short period of time by shocking the acne molecules in the matured bread and pores, splitting them into smaller particles and then clearing them out of the skin. The whole process is painless.

B、Clearance of hair rash (as in Figure B)

With a vibration frequency of more than 25,000 times per second, it can quickly remove hair rashes and cosmetic residues from pores.

C、Cleansing of the skin (as in Figure C)

The spatula head quickly and effectively removes dirt and aged keratin from the surface of the skin.

D、Introduce massage (as in Figure D)

The spatula head is placed flat on the surface of the skin in a pulling motion (as shown in Figure D), which better conducts the serum and other treatments into the skin.

Aftercare:

- **GENTLE CLEANSING:** Rinse your face with water or use a mild cleanser to wash off any remaining dirt.
- **Pore tightening & soothing:**
 - **Ice/cold spray:** Apply the ice hammer that comes with the device to your face or spray soothing mineral water to help shrink pores.
 - **Astringent Lotion/Soothing Mask:** Moisten with an astringent lotion containing witch hazel, tea tree, aloe vera, B5, etc. or apply a soothing moisturizing mask.
- **Moisturize and repair:** Follow up with simple ingredients, moisturizing and repairing serums and lotions/creams. Avoid harsh products


Cleaning Instruments

Rinse the shovel head with water and dry it with a soft cloth to keep it dry.

CAVEAT

- Do not hold the working spatula head stationary on one part of the skin to avoid damaging the skin.

- When using the spatula to clean the skin, pay attention to the strength of the use of the spatula, the spatula head and the skin at 45 degrees is optimal (as shown in the picture on the right).
- To use the spatula for an introductory massage, the spatula head is placed flat on the surface of the skin and the handle of the spatula is at 20 degrees to the skin (as shown in the picture).

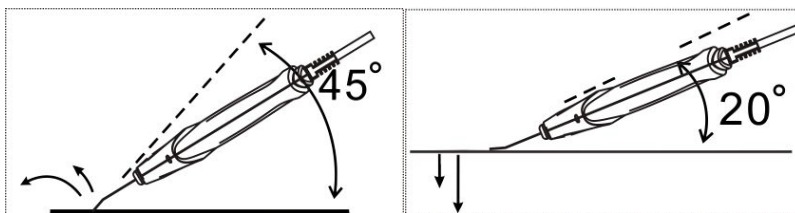

- When the skin surface is not sufficiently moisturized by the cleansing serum, it should be replenished.
- Spatulas should be used gently, after each use must be cleaned on the surface of the spatula head of dirty things, but not put the spatula in the water to wash, clean and dry disinfection, keep clean and dry.

Specifications and technical parameters

Product name	Multi-function beauty instrument
Function	Ultrasonic、 spray water oxygen、 aqua cleaning、 poration、 cold and hot hammer、 skin scrubber
Rated voltage	AC 100-120V~ AC 220-240V~
Rated working	50/60Hz
Output power	30W
Size(length *width*height)	53x37x35cm
Net weight	6.18Kg

Thank you for purchasing this product!

- Please read the entire instruction manual carefully before using the product. Take note of any warnings included in this manual.
- Always keep this instruction manual handy and if you give the device to another person, hand over the manual as it is an essential part of the product.
- Some components of the product may differ from the image that appears in catalogues, websites, brochures, etc.
- In order to enhance the product, the manufacturer reserves the right to modify its features and/or components without prior notice.

WK

Imported by
WEELKO BARCELONA SL
Pol. Ind. La Valldan

Camí de Garreta, 92
08600 Berga
Barcelona - Spain
T. +34 93 824 90 20
E. info@weelko.com